CRYO1 ACCESSORY KIT INSTALLATION INSTRUCTIONS

OTS Nitrous Oxide Tank Heater

AtriCure Incorporated
7555 Innovation Way
Mason, Ohio 45040 USA
Customer Service:
1-866-349-2342 (toll free)
1-513-755-4100 (phone)
P000654 Rev F

European Representative:
Köntges SPRL
Avenue Helleveldt 35
B-1180 Brussels Belgium
Tel: +32 (0) 2 375 51 63
FAX: +32 (0) 2 375 89 06
e-mail: herbert.kontges@skynet.be

Table of Contents

	Page
1. About This Installation	3
2. CCS-200 Shut Down	4
3. Product Description	4
4. Installation Procedure	5
5. Operating Instructions	6
6. Shutdown Procedure	7
7. Verify Accessory Kit Installation	7
8. Care and Maintenance	7
9. Return of Product	9
10. Disclaimer Statements	9

Cryo1 Accessory Kit Installation Instruction

1. About This Installation

The Cryo1 Accessory Kit consists of three (3) unit installation. The installation of each unit is required for use of the AtriCure® Cryo1 cryo-ablation probe with the frigitronics® CCS-200 Cardiac Cryosurgical System (CCS-200). This kit must be installed by personnel properly trained on the CCS-200.

The three units can be installed in any order:

1. CPR1 – Cylinder Pressure Regulator Unit
2. CPT1 – Cylinder Pressure Temperature Display Unit
3. OTS1/OTS2 – Nitrous Oxide Tank Heater

WARNINGS

The Cryo1 accessory kit is for use with the frigitronics CCS-200 Cardiac Cryosurgical System and Cryo1-cryo ablation probe.

The Cryo1 accessory kit is used to maintain appropriate operating pressure levels in the compressed nitrous oxide gas cylinders of the frigitronics CCS-200 Console.

Do not use with gas cylinders other than nitrous oxide.

CAUTIONS

Read all instructions carefully for the Cryo1 accessory kit prior to installing the components. Failure to properly follow instructions may lead to injury and may result in improper functioning of the accessory components.

Use of the Cryo1 accessory kit should be limited to properly trained and qualified medical personnel.

To avoid damage to the device, do not drop or toss any of the Cryo1 accessory components.

Do not use the Cryo1 components if damaged in any way.

Do not remove or install Cryo1 accessory components unless the line and exhaust pressure gauge read "0" psi on the CCS-200 console.

Do not restrict, kink, bend, clamp or otherwise damage nitrous oxide lines.

Getting Help

If you need help with this procedure please contact AtriCure, Inc. Customer Support at 1-877-492-1066.

2. CCS–200 Shut Down

NOTE: For additional help on shutting down the system, please refer to the *frigitronics CCS-200 Operation and Maintenance Manual*.

1. Turn the Nitrous Oxide **Cylinder Valves** clockwise to the “OFF” position.

Turn the system **On/Off Valves** “ON” and “OFF” repeatedly until all pressure gauges read “0” psi and the Exhaust Pressure Gauge needles touch the silver stop.

2. Turn system **On/Off Knobs** to the “OFF” position.

CAUTION: Do not remove or install probes unless the line and exhaust pressure gauge read “0” psi.

3. If attached, remove probe(s).
4. Turn **Main Meter switch** to the “OFF” position.
5. The CCS–200 shut down is complete.

3. Product Description

Important Preparation Notes:

It is recommended that the Tank Heater be installed and activated approximately

ONE (1) hour prior to beginning of the surgical procedure.

Initial tank pressure range must be 925 - 950 psi. DO NOT allow tank pressure to exceed 950 psi.

If the pressure exceeds 950 psi, unplug the Tank Heater from the outlet. If tank pressure drops below 925 psi prior to the beginning of the procedure, plug the Tank Heater back into the power outlet until the starting pressure reaches 925- 950 psi.

This device is intended for use only with the AtriCure Frigitronics CCS200 Console.

1. The Tank Heater is temperature controlled (pre-set to 34°C).
2. Each Tank Heater consists of three (3) components to be assembled. See **Figure 1**.

- Item 1 – **Band** (includes spring retainers and a connector)
- Item 2 – **Power Cord** (includes a connector)
- Item 3 – **Controller** (includes two receptacles for connectors)

Figure 1 - Tank Heater components prior to assembly

4. **Installation Procedure**

1. Loosen the tank restraint strap of the **CCS200** console.
2. Wrap the band around the N₂O tank with connector facing upwards.
3. **Secure ALL (6) spring retainers** as shown in **Figure 2**.

Figure 2 – Secure spring retainers, top and bottom retainers first.

4. Re-tighten the tank restraint strap. See **Figure 3a**.
5. If not already assembled, connect the controller to the power cord and the band. See **Figure 3b**.
6. Connect the **CCS200** inlet line to the N₂O tank valve. **Open tank valve.**

Figure 3a- Tank restraint strap

Figure 3b – Assembled components

5. Operating Instructions

1. Verify that the **ON / OFF** control valves on the **CCS200** console are each set to the **“OFF”** position. Check both System 1 and System 2 control valves.

CAUTION: If Tank Heater is plugged in for more than 30 minutes with the tank valve **CLOSED**, **OPEN** tank valve **SLOWLY**. The Console may be damaged if pressure exceeds 950 psi. Unplug the Tank Heater from the power outlet. Wait two (2) hours minimum before opening the valve.

2. Plug the Tank Heater power cord into a 110V outlet. The controller will always be **“ON”**.
3. **Monitor** the tank pressure gage on the **CCS200** filter assembly every 15 minutes.
4. If the **tank pressure** reaches **950 psi**, **unplug the Tank Heater**.
5. Continue to **monitor** the cylinder pressure gauge every 15 minutes.

6. After the surgical procedure starts, the Tank Heater should be left **“ON”** (plugged in) *unless the cylinder pressure exceeds 950 psi*. Continue to monitor the pressure gauge.
 7. If the **tank pressure** reaches **950 psi**, **unplug the Tank Heater**.
 8. Continue to **monitor** the cylinder pressure gauge **every 15 minutes**.
 9. After the surgical procedure starts, the Tank Heater should be left **“ON”** (plugged in) *unless the cylinder pressure exceeds 950 psi*. Continue to monitor the pressure gauge.
-

6. Shutdown Procedure

NOTE: Tank Heater may be very warm. Let tank and Tank Heater cool prior to removal.

1. Unplug the Tank Heater power cord from the power outlet.
 2. Close the N₂O tank valve.
 3. Bleed remaining pressurized N₂O from the CCS200 system. Follow procedure located on top console.
 4. Disconnect the **CCS200** inlet line from the N₂O tank valve.
 5. Loosen the tank restraint straps and open the spring retainers.
 6. Remove the Tank Heater.
 7. Re-tighten the tank restraint strap.
-

7. Verify Accessory Kit Installation

WARNING: Installation of all three (3) of the Cryo1 Accessory Components is required to the CCS-200 prior to using the AtriCure Cryo1 cryo-ablation probe. This installation Instruction addresses only the Nitrous Oxide Tank Heater.

8. Care and Maintenance

1. The CCS-200, including Cryo1 accessory components, should be handled with the care appropriate to any precision medical instrument. Avoid banging, dropping or mishandling the system. Periodically inspect the hoses and fittings for any damage or wear. Other than maintaining an adequate supply of medical grade nitrous oxide, the system is essentially maintenance free.

2. Cleaning:
NOTE: Do not spray or pour liquids directly on the unit.
NOTE: The unit and/or accessories cannot be sterilized.
WARNING: Ensure Isopropyl Alcohol (IPA) is completely dry before operating the unit.
CAUTION: Avoid caustic or abrasive cleaners

Guidelines

The following guidelines are recommended for cleaning the unit. It is the user's responsibility to qualify any deviations from these processing methods.

1. Disconnect the unit or cart from the outlet before cleaning.
2. If the unit and/or accessories are contaminated with blood or other body fluids, they shall be cleaned before the contamination can dry (within two hours of contamination).
3. The outer surfaces of the unit and/or accessories shall be cleaned with 70% -90% Isopropyl alcohol (IPA) wipes for a minimum of two minutes. Do not allow fluids to enter the chassis.
4. Pay attention to all areas where fluids or soil may gather, such as under/ around the handles or any tight crevices/ grooves.
5. Dry the unit and/or accessories with a dry, white lint-free cloth.
6. Conduct a final confirmation of the cleaning process by visually inspecting the white cloth for remaining soil.
7. If soil remains on the white cloth, repeat steps 3 through 6.
8. After cleaning is complete, turn the unit on to perform Power On Self-Test (POST). If any errors are received, contact AtriCure to begin return process.

9. Return of Product

If for any reason this product must be returned to AtriCure, Inc., a return goods authorization (RGA) number is required from AtriCure, Inc., prior to shipping.

CAUTION: It is the responsibility of the health care institution to adequately prepare and identify the product for shipment.

If the product has been in contact with blood or body fluids, it must be thoroughly cleaned and disinfected before packing. It should be shipped in either the original carton or an equivalent carton, to prevent damage during shipment; and it should be properly labeled with an RGA number and an indication of the bio-hazardous nature of the contents of shipment.

Instructions for cleaning and materials, including appropriate shipping containers, proper labeling, and an RGA number may be obtained from AtriCure, Inc.

10. Disclaimer Statements

Users must assume responsibility for approving the condition of the installation before it is used. AtriCure, Inc., cannot be held liable for any consequential damage, personal injury or damage to property caused by improperly installed products.

This document describes the proper procedures for the product installation. Any deviation from these procedures, which may compromise the function of the product, is the responsibility of the user.

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