## **CRYO1 ACCESSORY KIT INSTALLATION INSTRUCTIONS**

### **Cylinder Pressure Temperature Display**

AtriCure Incorporated  
7555 Innovation Way  
Mason, Ohio 45040 USA  
Customer Service:  
1-866-349-2342 (toll free)  
1-513-755-4100 (phone)  
P000623 Rev E

European Representative:  
Köntges SPRL  
Avenue Hellevelt 35  
B-1180 Brussels Belgium  
Tel: +32 (0) 2 375 51 63  
FAX: +32 (0) 2 375 89 06  
e-mail: herbert.kontges@skynet.be

# Table of Contents

	Page
1. About This Installation	3
2. CCS-200 Shut Down	4
3. Replacement versus First Time Installation	4
4. Cylinder Pressure Temperature Display Removal	4
5. Cylinder Pressure Temperature Display Installation	5
6. Verify Accessory Kit Installation	7
7. Care and Maintenance	7
8. Return of Product	8
9. Disclaimer Statements	8

# **Cryo1 Accessory Kit Installation Instruction**

## **1. About This Installation**

The Cryo1 Accessory Kit consists of three (3) unit installation. The installation of each unit is required for use of the AtriCure® Cryo1 cryo-ablation probe with the frigitronics® CCS-200 Cardiac Cryosurgical System (CCS-200). This kit must be installed by personnel properly trained on the CCS-200.

The three units can be installed in any order:

1. CPR1 – Cylinder Pressure Regulator Unit
2. CPT1 – Cylinder Pressure Temperature Display Unit
3. OTS1/OTS2 – Nitrous Oxide Tank Heater

## **WARNINGS**

The Cryo1 accessory kit is for use with the frigitronics CCS-200 Cardiac Cryosurgical System and Cryo1-cryo ablation probe.

The Cryo1 accessory kit is used to maintain appropriate operating pressure levels in the compressed nitrous oxide gas cylinders of the frigitronics CCS-200 Console.

Do not use with gas cylinders other than nitrous oxide.

## **CAUTIONS**

Read all instructions carefully for the Cryo1 accessory kit prior to installing the components. Failure to properly follow instructions may lead to injury and may result in improper functioning of the accessory components.

Use of the Cryo1 accessory kit should be limited to properly trained and qualified medical personnel.

To avoid damage to the device, do not drop or toss any of the Cryo1 accessory components.

Do not use the Cryo1 components if damaged in any way.

Do not remove or install Cryo1 accessory components unless the line and exhaust pressure gauge read "0" psi on the CCS-200 console.

Do not restrict, kink, bend, clamp or otherwise damage nitrous oxide lines.

## **Getting Help**

If you need help with this procedure please contact AtriCure, Inc. Customer Support at 1-877-492-1066.

---

## 2. **CCS-200 Shut Down**

**NOTE:** For additional help on shutting down the system, please refer to the *frigitronics CCS-200 Operation and Maintenance Manual*.

1. Turn the Nitrous Oxide **Cylinder Valves** clockwise to the “OFF” position.  
Turn the system **On/Off Valves** “ON” and “OFF” repeatedly until all pressure gauges read “0” psi and the Exhaust Pressure Gauge needles touch the silver stop.
2. Turn system **On/Off Knobs** to the “OFF” position.

**CAUTION:** Do not remove or install probes unless the line and exhaust pressure gauge read “0” psi.

3. If attached, remove probe(s).
4. Turn **Main Meter switch** to the “OFF” position.
5. The CCS-200 shut down is complete.

---

## 3. **Replacement versus First Time Installation**

**NOTE:** If this is a first time installation of the Cylinder Pressure Temperature Display, skip Section 4 and proceed directly to Section 5.

---

## 4. **Cylinder Pressure Temperature Display Removal**

**NOTE:** Lock the casters to the CCS-200 console prior to proceeding with the removal of the Cylinder Pressure Temperature Display Unit to prevent cart motion.

1. With both hands, firmly grasp the Cylinder Pressure Temperature Display at the upper corners of the display module as highlighted below.
2. Push sharply in the direction shown below to cause Velcro separation between the Cylinder Pressure Temperature Display unit and the CCS-200 console surface.
3. Package the defective Cylinder Pressure Temperature Display unit in the box that the replacement unit was delivered, and return to customer service with an RGA number for service.

## 5. *Cylinder Pressure Temperature Display Installation*

Cylinder Pressure Temperature Display Kit		
Item	Description	Quantity
1.	Temperature Display (with velcro pads)	1
2.	Alcohol Wipe Pad	1

Confirm that all items listed above are in the Cylinder Pressure Temperature Display Kit.

**NOTE:** The Cylinder Pressure Temperature Display Unit must be installed on the same CCS-200 that the Cylinder Pressure Regulator is on.

1. The CCS-200 top console area must be cleaned with the provided isopropyl alcohol wipe before mounting the Temperature Display Unit.

**CAUTION:** You only have one opportunity to attach the Cylinder Pressure Temperature Display Unit properly. Confirm the proper placement and alignment before removing the red tape liner from the bottom of the Cylinder Pressure Temperature Display Unit.

2. Remove the adhesive backing of the Velcro pad from the Cylinder Pressure Temperature Display Unit.
3. Locate the center of the CCS-200 console, approximately in line with the Tissue Temperature input connector locale.

4. Align the Cylinder Pressure Temperature Display Unit over the center and in line with the front edge of the CCS-200 console top.

NOTE: The red and black connectors face the front of the CCS-200 console.

5. Once comfortable with the alignment, carefully adhere the unit to the CCS-200 console.
6. Apply firm pressure to the Cylinder Pressure Temperature Display Unit to ensure a strong bond.
7. The Cylinder Pressure Temperature Display Unit Installation is complete.

---

## 6. Verify Accessory Kit Installation

**WARNING:** Installation of all three (3) of the Cryo1 Accessory Components is required to the CCS-200 prior to using the AtriCure Cryo1 cryo-ablation probe. This installation Instruction addresses only the Cylinder Pressure Temperature Display Module.

---

## 7. Care and Maintenance

1. The CCS-200, including Cryo1 accessory components, should be handled with the care appropriate to any precision medical instrument. Avoid banging, dropping or mishandling the system. Periodically inspect the hoses and fittings for any damage or wear. Other than maintaining an adequate supply of medical grade nitrous oxide, the system is essentially maintenance free.

2. Cleaning:

**NOTE: Do not spray or pour liquids directly on the unit.**

**NOTE: The unit and/or accessories cannot be sterilized.**

**WARNING: Ensure Isopropyl Alcohol (IPA) is completely dry before operating the unit.**

**CAUTION: Avoid caustic or abrasive cleaners**

### Guidelines

The following guidelines are recommended for cleaning the unit. It is the user's responsibility to qualify any deviations from these processing methods.

1. Disconnect the unit or cart from the outlet before cleaning.
2. If the unit and/or accessories are contaminated with blood or other body fluids, they shall be cleaned before the contamination can dry (within two hours of contamination).
3. The outer surfaces of the unit and/or accessories shall be cleaned with 70% -90% Isopropyl alcohol (IPA) wipes for a minimum of two minutes. Do not allow fluids to enter the chassis.

4. Pay attention to all areas where fluids or soil may gather, such as under/ around the handles or any tight crevices/ grooves.
5. Dry the unit and/or accessories with a dry, white lint-free cloth.
6. Conduct a final confirmation of the cleaning process by visually inspecting the white cloth for remaining soil.
7. If soil remains on the white cloth, repeat steps 3 through 6.
8. After cleaning is complete, turn the unit on to perform Power On Self-Test (POST). If any errors are received, contact AtriCure to begin return process.

---

## **8. Return of Product**

If for any reason this product must be returned to AtriCure, Inc., a return goods authorization (RGA) number is required from AtriCure, Inc., prior to shipping.

**CAUTION:** It is the responsibility of the health care institution to adequately prepare and identify the product for shipment.

If the product has been in contact with blood or body fluids, it must be thoroughly cleaned and disinfected before packing. It should be shipped in either the original carton or an equivalent carton, to prevent damage during shipment; and it should be properly labeled with an RGA number and an indication of the bio-hazardous nature of the contents of shipment.

Instructions for cleaning and materials, including appropriate shipping containers, proper labeling, and an RGA number may be obtained from AtriCure, Inc.

---

## **9. Disclaimer Statements**

Users must assume responsibility for approving the condition of the installation before it is used. AtriCure, Inc., cannot be held liable for any consequential damage, personal injury or damage to property caused by improperly installed products.

This document describes the proper procedures for the product installation. Any deviation from these procedures, which may compromise the function of the product, is the responsibility of the user.

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