25. Press the activation button to engage FREEZE Mode for the desired length of time. The system will automatically cycle from FREEZE to DEFROST after the Ablation Time has expired.

WARNING: Use care to avoid PROBE movement while cryoablation is present, to prevent inadvertent tissue damage.

CAUTION: When using a standard off-the-shelf nerve stimulator, read all of the manufacturer's instructions carefully prior to using the device. Failure to follow instructions may lead to injury and may result in improper function of the device.

CAUTION: The care while the CONSOL is in Defrost Mode, as during N/O gas wasting, the PROBE may not sufficiently cool cryoablation.

CAUTION: Ensure the CONSOL is in Ready Mode before attempting to disconnect the PROBE. The sudden release of pressurized gas may cause the PROBE to recoil, which may injure the operator or patient.

WARNING: Do not depress or reuse the PROBE. These can cause patient injury and/or the communication of infectious diseases from one patient to another.

DISPOSAL: Upon completion of the surgical procedure, after the device has been disconnected from the CONSOL, discard the PROBE. Follow local governing authorities and recycling plans regarding disposal or recycling of the PROBE.

RETURN OF USED PRODUCT: If for any reason this product may be returned to Atcryo, Inc., a return goods authorization (RGA) number is required from Atcryo, Inc. This RGA number must be included on all return packaging. It must be properly cleaned and disinfected before shipment. It should be shipped either in original cart or an equivalent carton, to prevent damage during transport, and should be properly labeled with an RGA number and an indication of the hazardous nature of the contents of shipping instructions for cleaning and materials, including appropriate shipping containers, proper labeling, and an RGA number may be obtained from Atcryo, Inc.

DISCLAIMER STATEMENTS: Users are responsible for approving the acceptable condition of this product before it is used, and for ensuring that the product is only used in the manner described in these instructions for use, including but not limited to, ensuring that the product is used as-reusable.

Table with 2 columns: PROBLEM and SOLUTION. Rows include issues like 'PROBE does not reach desired defect temperature after freeze' and 'PROBE does not reach proper temperature'.

CAUTION: Nitrous Oxide gas must be safely exhausted. Follow standard hospital guidelines for allowable concentration in the operating room.

CAUTION: Ensure the CONSOL is in Ready Mode before attempting to connect the PROBE. The sudden release of pressurized gas may cause the PROBE to recoil, which may injure the operator or patient.

CAUTION: Ensure the CONSOL is in Ready Mode (see Figure 6), connect the PROBE Connectors to the CONSOL Ports as follows (see Figure 2).

CAUTION: The distal end of the Rigid PROBE Shaft should not be bent more than 5 cm (2.0 inches) from straight, as shown in Figure 5.

CAUTION: Repetitive bends in the same location could damage the Malleable Section of PROBE causing device malfunction.

CAUTION: Do not use the PROBE if damaged or if it may result in device malfunction. The PROBE has a limited function if greater than 14 freeze/Defrost cycles are intended. It is recommended to use a second probe.

CAUTION: Do not use the PROBE if damaged or if it may result in device malfunction. The PROBE has a limited function if greater than 14 freeze/Defrost cycles are intended. It is recommended to use a second probe.

CAUTION: Repetitive bends in the same location could damage the Malleable Section of PROBE causing device malfunction.

CAUTION: Do not use excessive force when using the PROBE to avoid tissue damage. Avoid direct contact with PROBE with long to prevent potential risk of pneumothorax.

WARNING: Avoiding the internal nerve should be at least 2 cm from the distal end ranging a 4 cm from the base of the probe to prevent the risk of abdominal mode bulging.

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