FIGURE 1/FIGURA 1

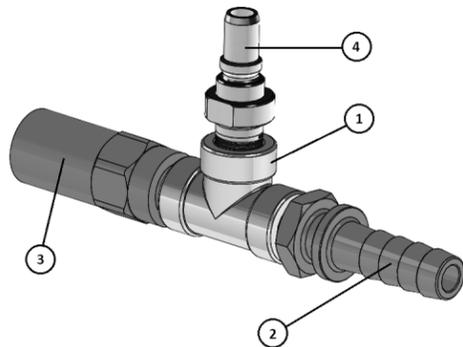
FIGURE 2/FIGURA 2

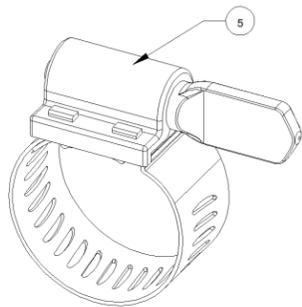
FIGURE 3/FIGURA 3

FIGURE 4/FIGURA 4

ACM Exhaust Hose Connector

INTENDED PURPOSE/INDICATIONS FOR USE

The intended purpose of the ACM Exhaust Hose Connector is an optional accessory of the AtriCure cryoICE BOX, providing a method to connect the AtriCure cryoICE BOX exhaust to a hospital medical vacuum or waste anesthesia gas disposal (WAGD) system. It is intended only to be used together with the AtriCure cryoICE BOX to enable meeting its intended purpose.

⚠️ **WARNING:** ⚠️

Please refer to the CryoICE Box (ACM) for Console Warnings, Cautions, product description, flow rates, and features. Care should be exercised in users with suspected or known allergies or hypersensitivity to nickel.

ACM EXHAUST HOSE CONNECTOR PERFORMANCE CHARACTERISTICS

The ACM Exhaust Hose Connector provides a method to connect the cryoICE BOX exhaust to a hospital Medical Vacuum or Waste Anesthesia Gas Disposal (WAGD) system.

ACM EXHAUST HOSE CONNECTOR CONFIGURATIONS

Part Number	Description	Region
A001150-15	Medical Vacuum Connector Chemetron by 1/4" MNPT - Brazil	BR
A001150-16	WAGD Connector Chemetron by 1/4" MNPT - Brazil	BR

Item Number	Description
1	Tee Fitting
2	Barbed Fitting
3	Pressure Relief Valve
4	Hospital Specific Connection
5	Hose Clamps

ACM EXHAUST HOSE CONNECTOR INSTALLATION

- Place two hose clamps (item 5) over the clear exhaust tubing.
- Push one end of the clear tubing over the barbed fitting on the back of the cryoICE BOX.
- Push the other end of the clear tubing over the barbed fitting (item 2) on the ACM Exhaust Hose Connector.
- Tighten the two hose clamps and verify both ends of the AtriCure cryoICE BOX clear exhaust tubing are securely attached. This confirms the exhaust hose connector is properly installed.

⚠️ **CAUTION:** The exhaust hose can come off the barbs during the freeze cycle if the hose is not secured properly. Connect the hospital specific connection (item 4) to a dedicated Medical Vacuum or WAGD Port.

⚠️ **WARNING:** ⚠️

The ACM Exhaust Hose Connector requires a dedicated Vacuum or WAGD Port to prevent back pressure into the patient's breathing line, which may result in a Pneumothorax.

**Preventative Maintenance:** No preventative maintenance required.

DISPOSAL OF RE-USABLE DEVICES:

- Disconnect the exhaust hose connector and treat as regulated medical waste requiring decontamination to render safe for further handling and disposal.
- Follow cleaning and disinfecting steps for the exhaust hose connector, as outlined in section 7 of the AtriCure cryoICE BOX IFU.
- Contact local medical equipment recycling and disposal service.

**Serious Incident Statement:** Any serious incident that has occurred in relation to this device should be reported to AtriCure and the competent authority of the Member State in which the user and/or patient is located.

**Disclaimer:** Under no circumstances will AtriCure, Inc. be responsible for any incidental, special or consequential loss, damage, or expense, which is the result of the deliberate misuse of this product, including any loss, damage, or expense which is related to personal injury or damage to property.

EXPLANATION OF SYMBOLS ON PACKAGE LABELING

Refer to the outer package label to see which symbols apply to this product.

⚠️	Caution	⚙️	Manufacturer	BR REP	Authorized representative in the Brazilian Community
🛡️	Segurança	🇧🇷	Segurança	🇧🇷	National Institute of Metrology Standardization and Industrial Quality

⚙️ AtriCure Inc.  
7555 Innovation Way  
Mason, Ohio 45040 USA  
+1 866 349 2342  
+1 513 755 4100

**Technical Name:** Cryogenic Surgical Unit  
**Commercial Name:** AtriCure Cryosurgical System  
**ANVISA n°:** 80117580971  
**Importer:** Emergo Brazil Import Importação e Distribuição de Produtos Médicos Hospitalares Ltda. Avenida Francisco Matarazzo, 1.752, Salas 502/503, Agua Branca, São Paulo-SP, CEP – 05001-200 CNPJ: 04.967.408/0001-98  
**Email:** brazilvigilance@ul.com  
**Technical Manager:** Luiz Levy Cruz Martins, CRF-SP: 42415

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