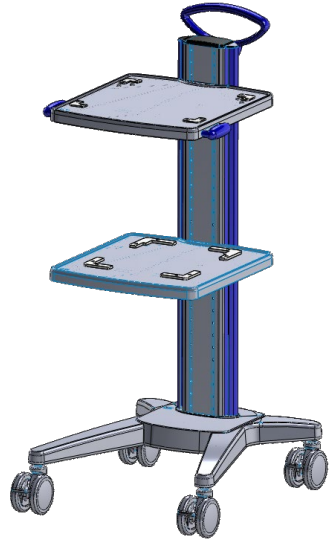
# AtriCure®

## Fusion Cart

Product Code: ASCF

**CAUTION:** Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

Information in this document is subject to change without notice. Reproduction of this document in any manner whatsoever, without the written permission of AtriCure®, Inc. is strictly forbidden. AtriCure® and the AtriCure® Logo are trademarks of AtriCure®, Inc. AtriCure® disclaims proprietary interest in the marks and names of others.

**Manufactured by:**  
**AtriCure Incorporated**  
 7555 Innovation Way  
 Mason, Ohio 45040 USA  
 Customer Service: Customer Service  
 Tel: 866.349.2342  
 1-866-349-2342 (toll free)  
 1-513-755-4100 (phone)

## Fusion Cart

The AtriCure Fusion Cart provides support and mobility for the Estech COBRA Electrosurgical Unit and a Medela Basic or Basic30 Vacuum Pump.

### The Fusion Cart features:

- 1160mm (46") Total Height
- 450mm (18") Width
- 686mm (27") Depth
- Ø100mm (4") Casters which are omni-directional, lockable casters, allowing for quick and easy transport of the system within hospitals and operating theatres.
- Top Shelf for supporting Estech ESU.
- Bottom Shelf for supporting Medela Basic or Basic30 vacuum pump.
- Cart Load Capacity: 65 kg (143 lbs)

### System Description:

The Fusion Cart supports the following AtriCure equipment:

Estech Bipolar System:

- Estech COBRA Electrosurgical Unit supplies radiofrequency energy in the range of 450-470 kHz. It operates in a temperature controlled, power limited manner, based on operator setting and temperature feedback provided by thermocouples in the devices.

Medela Vacuum pump

- Medela Basic or Basic30 pumps are suction pumps powered by a piston/cylinder system and provides maximum suction performance for many applications.

### Indications for Use:

The AtriCure Fusion Cart is intended to be used as an accessory to the Estech Generator and Medela pump.

No part of the AtriCure Fusion Cart or its accessories is intended to make contact with the patient.

### Equipment Classification:

The AtriCure Fusion Cart is categorized as: Class 1 Equipment; Type CF Applied Part; IPX0.

### Environmental Indications:

NOTE: Environmental Conditions: (1) operational temperature +10° C to +40° C; (2) storage temperature -35° C to +54° C; and (3) humidity 15% to 90% relative humidity.

### Proper Use:

**NOTE:** The AtriCure Fusion Cart is ONLY intended for use with the equipment listed under the System Description.

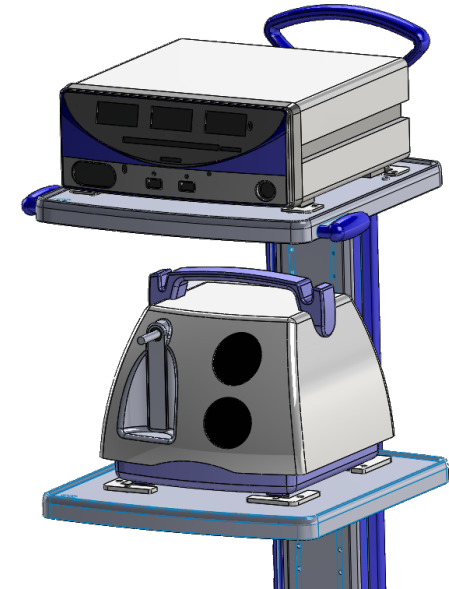
**NOTE:** The cart should never be used to carry equipment other than what is specified in these instructions. The cart should not be operated on an incline exceeding 10°. During operation and storage, the casters should be locked securely.

**NOTE:** No modifications to this cart are allowed as this can create a potential for safety issues. Shelf locations should not be modified.

**NOTE:** These instructions do not address specific instructions related to the equipment listed under System Description. The user should refer to product specific manuals provided with individual components.

### Equipment Placement:

1. Place Estech COBRA electrosurgical unit on top shelf. Verify feet of unit are internal to nesting brackets.
2. Place Medela Basic or Basic30 Vacuum pump on bottom shelf. Verify feet of unit are internal to nesting brackets.

### Cleaning Instructions

NOTE: The Fusion Cart Cleaning and Disinfection Instructions: (1) use a mild detergent (prepared to its specifications) and a damp cloth; and (2) do not use caustic, corrosive, or abrasive cleaning materials.

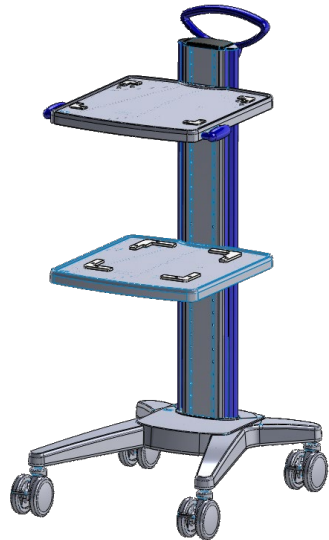
### Serviceable Parts

NOTE: The Fusion Cart does not have any serviceable parts. For servicing issues, contact AtriCure, Inc.

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Observação: O carrinho Fusion não tem peças reparáveis. Para assuntos de reparo, entre em contato com a AtriCure, Inc.