# Tender LiV™ Care (TLC) Non Sterile Reusable Devices


## TABLE OF CONTENTS:

English .....	2
Dutch .....	4
French.....	6
German .....	8
Italian .....	10
Spanish.....	12

**AtriCure Incorporated**  
7555 Innovation Way  
Mason, Ohio 45040 USA  
Customer Service:  
1-866-349-2342 (toll free)  
1-513-755-4100 (phone)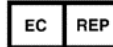

**Köntges SPRL**  
Avenue Hellevelt 35  
B-1180 Brussels Belgium  
Tel: +32 (0) 2 375 51 63  
FAX: +32 (0) 2 375 89 06  
Email: herbert.kontges@skynet.be

BEFORE THE PROCEDURE

DURING THE PROCEDURE

AFTER THE PROCEDURE

Storage  
Transport

Preparation  
Storage

Cleaning, Lubrication,  
and Sterilization

Reference: Instructions for Reprocessing of Reusable Devices, P/N 440-11975-02

**Before the Procedure**

- Store instruments in the proper tray at all times
  - Transport instruments in tray to avoid damage
  - Test function prior to use
- Contact Customer Service for replacements

 <p>Manufactured by: AtriCure Incorporated 7555 Innovation Way Mason, Ohio 45040 USA Customer Service: 1-866-349-2342 (toll free) 1-513-755-4100 (phone)</p>	<table border="1" style="width: 100%;"> <tr> <td style="width: 50%; text-align: center;">EC</td> <td style="width: 50%; text-align: center;">REP</td> </tr> </table> <p>European Representative: Köntges SPRL Avenue Helleveldt 35 B-1180 Brussels Belgium Tel: +32 (0) 2 375 51 63 FAX: +32 (0) 2 375 89 06 e-mail: herbert.kontges@skynet.be</p>	EC	REP
EC	REP		

**During the Procedure**

- Flush instruments with saline periodically
- DO NOT ALLOW BLOOD TO DRY ON INSTRUMENTS

**After the Procedure**

CAUTION! 

- **Avoid caustic cleaners**  
*Cleaning solutions should be pH neutral*
- **Avoid automated washers**  
*Instruments should be hand washed*
- **Avoid tap water**  
*Use DI or RO water only*
- **Avoid wet loads**  
*Instruments must be dry when stored*

**Prior to Sterile Processing Department (SPD)**

- Hand rinse and flush with sterile water
- Send to Sterile Processing Department (SPD) immersed in sterile water

**Cleaning at Sterile Processing Department (SPD)**

- Fill sink with Enzymatic Cleaner and de-ionized (DI) or reverse osmosis filtered (RO) water to manufacturer specifications
- Immerse entire instrument in the sink
- Using syringe with flush port adapter attached, flush instrument 2x or until no debris is seen exiting the instrument
- Using soft cleaning brush, scrub jaws of instrument until clean
- Actuate instrument to assure free movement and that all spots are clean
- Rinse with DI or RO water
- Flush with DI or RO water 2x or until no cleaner is present
- Using air hose and compressed air, blow instrument dry (especially down the flush port)

**Lubrication**

- Lubrication of the device using Steris® Hinge-Free® instrument lubricant (or equivalent) per manufacturer's instructions is recommended prior to each device sterilization cycle.

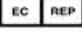
**Steam Sterilization**

- Please be sure to use DI or RO water
- Pre-Vacuum, Steam Sterilization - 132 °C (270 °F) to 137 °C (278.6 °F) 4 Minutes (recommended)
- Pre-Vacuum, Steam Sterilization - 134°C (275 °F) to 137 °C (278.6 °F) 3 Minutes

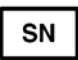
Keep instruments in the proper tray at all times for transport to avoid damage - DO NOT STACK INSTRUMENTS ON EACH OTHER

**Disposal Method**

- Follow local governing ordinances and recycling plans regarding or recycling of device components.

 Manufactured by: AtriCure Incorporated 7555 Innovation Way Mason, Ohio 45040 USA Customer Service: 1-866-349-2342 (toll free) 1-513-755-4100 (phone)	 European Representative: Köntges SPRL Avenue Helleveldt 35 B-1180 Brussels Belgium Tel: +32 (0) 2 375 51 63 FAX: +32 (0) 2 375 89 06 e-mail: Herbert.kontges@skynet.be
---	---

**Graphic Symbols for Device Labeling**

Catalogue Number 	Serial Number 	Quantity 	Date of Manufacture 
Manufacturer/Company Address 	Authorized Representative in the European Union 	Instrument supplied Non-Sterile 	CE mark, Product conforms with the essential requirements in the European Medical Devices Directive 93/42/EEC. 
Does not contain Natural Rubber Latex 	Contains no di (2-ethylhexyl) phthalate (DEHP) released from polyvinyl chloride (PVC) 	Consult Instructions For Use 	Caution: U.S. Federal law restricts this device to sale by or on the order of a physician or other licensed practitioner. 

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