


# ORLab Setup Quick Reference Guide




01 / Plug electrical cord into outlet.

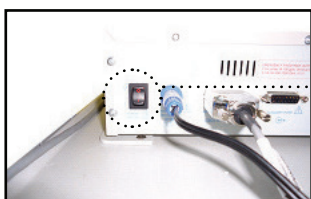


1

02 / Starting at the bottom of the ORLab Turn “ON” the switches in the following order. “1” Simulating Generator Unit.

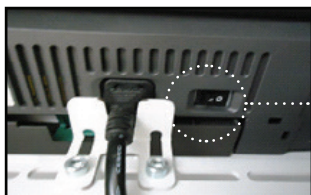

2


03 / “2” Next, turn on the printer.

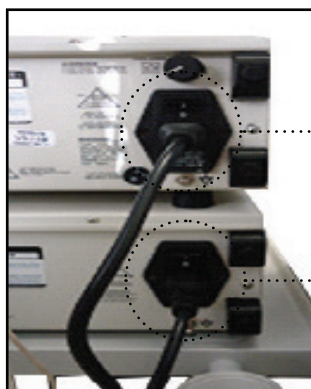

4


04 / Turn on the AtriCure RF generators, (refer to picture, “3” then “4”).

3

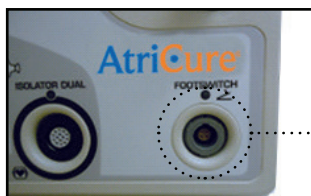

5

05 / Now move to the front of the ORLab and plug in the foot pedal (located on side of ORLab) into the designated area (“5”) and place next to surgeon’s foot.

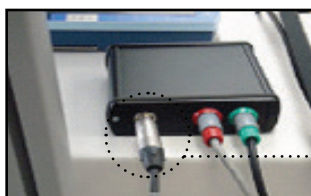


6

06 / Plug in ECG Slave Cable into available ECG outlet (located behind perfusion pump) (“6”).




7

07 / Press button to turn “computer” to “ON” position (“7”).

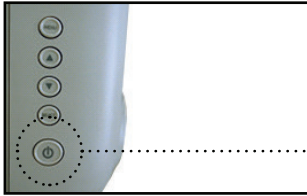

8

08 / Press button to turn “touch screen” to “ON” position (“8”).

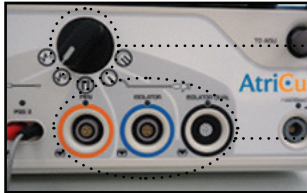

9

09 / Plug in AtriCure Clamp or Pen into corresponding port (“9”) and toggle switch to desired Bipolar Clamp or Pen position until green light is lighted above hand piece (“10”).

## Ablation Cycle

- 01 / Ask surgeon to press foot pedal when ready to “Ablate”.
- 02 / Ablation cycle will change tones every ten seconds.
- 03 / Ablation cycle is finished when tone goes a “steady beep” and transmural light flashes on front of generator.



**NOTE: Surgeon should then stop pressing foot pedal.**

- 04 / Repeat steps 1-3 as needed for respective hand pieces and additional ablations.
- 05 / To switch to other hand pieces, switch toggle “Black” switch to desired Bipolar Clamp or Pen position until green light is lighted above hand piece connection.

## Shut Down Process

- 01 / Disconnect Clamps/Pens from generator
- 02 / Switch off each individual unit starting at step 8 and working back to step 1 (as stated above in process).
- 03 / Disconnect ECG cable and stow onto ORLab.
- 04 / Disconnect foot pedal and stow onto ORLab.
- 05 / Unplug from wall and store.

### ATRICURE, INC.

7555 Innovation Way  
 Mason, OH 45040 USA  
 +1 (513) 755-4100  
 +1 (888) 347-6403  
 CustomerService@AtriCure.com  
 www.AtriCure.com

Please review the Instructions for Use for a complete listing of contraindications, warnings, precautions and potential adverse events prior to using these devices.

Rx Only.